Collection and transport system for microbiological samples

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 **swab**[®]

Range of δ swab® system Gama del sistema δ swab®

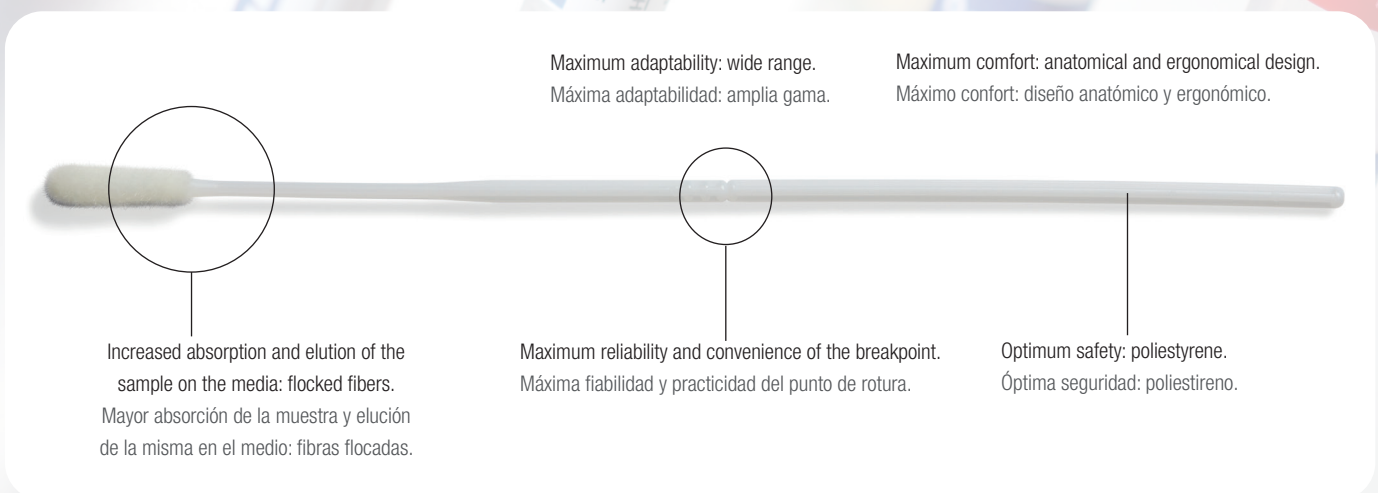
Range of swabs for collecting and transporting microbiological samples in liquid medium as well as a new line of enrichment media.

- Compatible with the new inoculation automation and sample streaking equipment.
- Compatible with molecular diagnostic techniques.
- Facilitates the collection, transport and subsequent treatment of microbiological samples.
- Greater reliability for sample recovery: high capacity for the absorption and elution of the sample from the flocked swab.
- Greater feasibility of the sample as it is totally suspended in the medium.
- Allows different homogeneous inocula to be obtained from the same sample.
- It adapts to any working protocol.
- Minimises any possible cross-contamination: less handling and maximum watertightness.
- More comfort for the user: conical base which facilitates stirring and greater stability of the tube with skirt base.
- More comfort for the patient, due to the softer swab covering.
- Standardises the receipt of samples at the microbiology labs.
- Facilitates manual streaking.
- Allows direct extension on the glass slide as the medium does not contain agar.
- Ensures proper sample transport and storage both at room temperatures (20°C - 25°C) and at refrigeration temperatures (4°C - 8°C).

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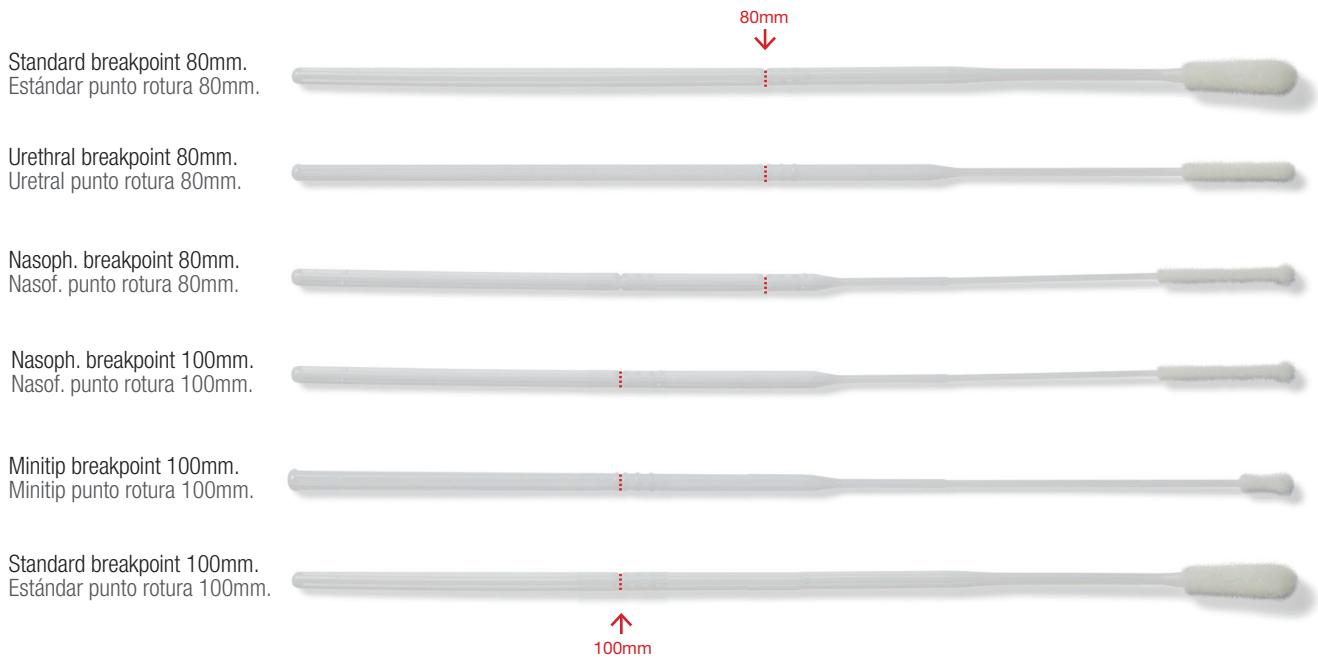
Optimum safety: polystyrene.
Óptima seguridad: poliestireno.

Deltalab offers a wide range of flocked swabs, with or without breaking point, that perform the most efficient absorption and elution for the collection of microbiological samples. These swabs are presented in tube or flow-pack without transport medium. For codes and features information, please ask our sales department.

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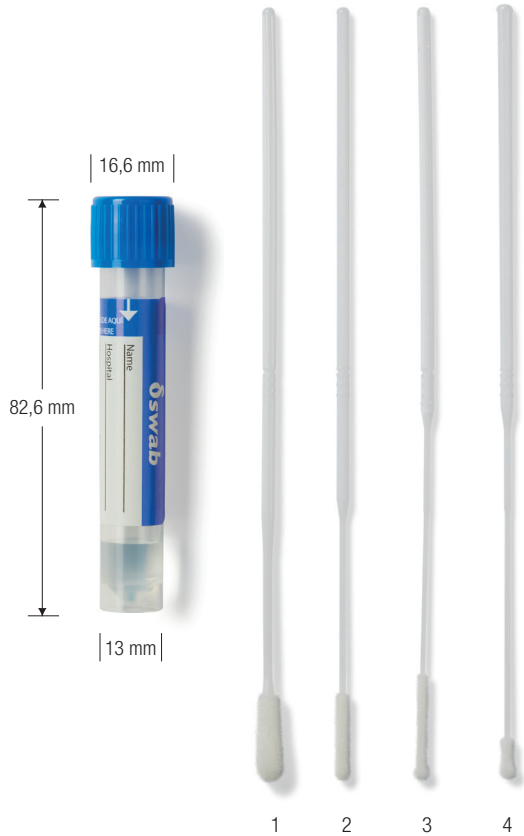
Particularly suited for microbiological sample collection and transport. It maintains the viability of Aerobes, Facultative Anaerobes and Anaerobes Bacteria for a minimum of 48 hours both at room temperature (20-25°C) and at refrigeration temperature (4-8°C) and 24 hours with Fastidious Bacteria.

Designed both for traditional and automated sample streaking. Compatible with the molecular diagnostic techniques and for direct extension on glass slides.

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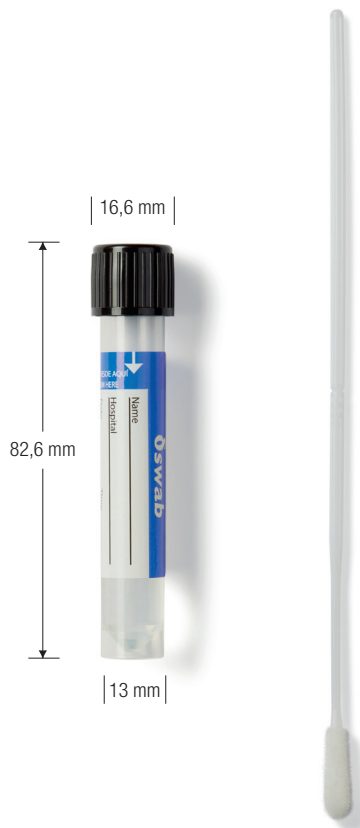
Specially suited for the collection and transport of faecal samples. Maintains the viability of the fecal pathogens for a minimum of 48 hours without overgrowth. Designed for carrying out sample collection directly from the rectum or from faeces.

Designed for traditional and automated sample streaking. Compatible with the molecular diagnostic techniques by PCR and for direct extension on glass slides.

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
Particularly suited for collecting and transporting microbiological samples which contain Virus, Chlamydia, Ureaplasma and/or Mycoplasma. Includes antibiotics which inhibit the growth of Bacteria and Fungi; this ensures reliability in terms of sample recovery. Maintains the infectability of viruses for a minimum of 96 hours both at room temperature (20-25°C) and at refrigeration temperature (4-8°C). Each tube contains glass beads to facilitate cell lysis, sample homogeneity and maximize the elution of the sample from the swab. Designed for cell cultivation and compatible with molecular diagnostic techniques by quantitative PCR.

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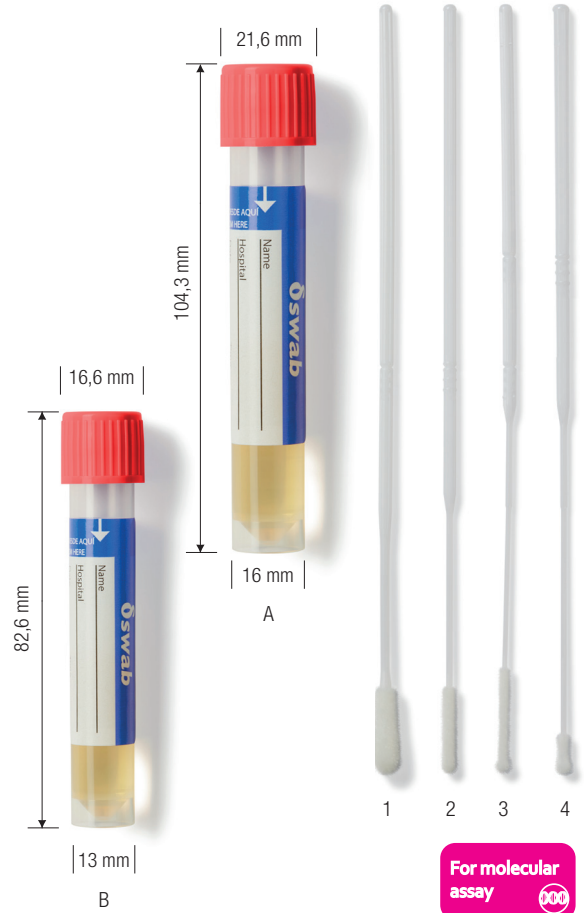Following the WHO recommendations, this product is indicated for the collection and transport of samples of viruses, like SARS-CoV-2. By including antibiotics that inhibit the growth of bacteria and fungi, it ensures suitability in sample recovery. It maintains the infectability of viruses for up to 96 hours, both at room temperature (20-25°C) and at refrigeration temperature (4-8°C). Designed for cell cultivation and to be compatible with the molecular diagnostic techniques by quantitative PCR.

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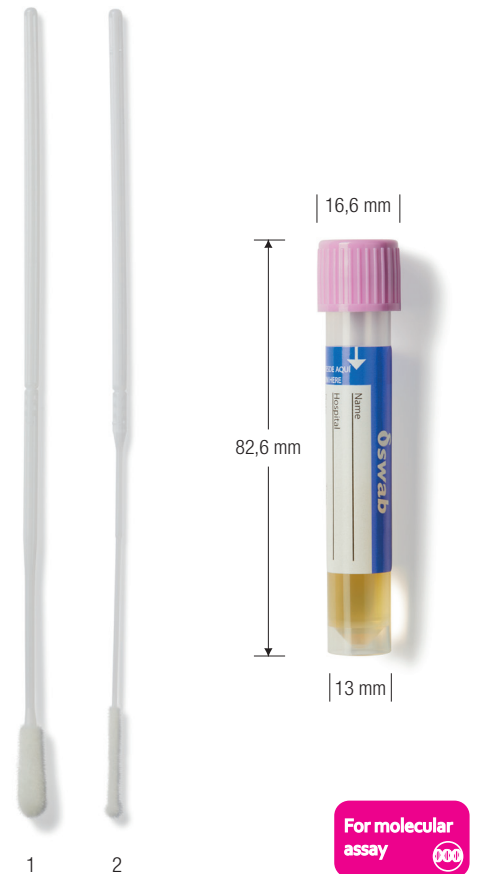
SELENITE SELENITO

IVD

Inhibits the proliferation of bacterial microbiote. It is particularly suited for isolating Salmonella from faecal samples, food and/or water. It is used as an enrichment media for clinical and industry samples.

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- For culture
- For automated systems
- For molecular assay
- Enrichment media

code código	ml ml	description descripción	case qty. cant. caja	case wt. peso caja	case vol. vol. caja	package presentación
304210	2 ml	Selenite broth Caldo Selenito	6 x 60	2.84	0.025	rack-case caja-rack

Cases per pallet · Cajas por palet: 32

SHELF LIFE / CADUCIDAD: 24 months/meses

THIOGLYCOLAT TIOGLICOLATO

IVD

Widely used as enrichment media for isolation and cultivation of Aerobic and Anaerobic Bacteria and Fastidious Bacteria. It is used as an enrichment media for clinical samples.

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- For culture
- For automated systems
- For molecular assay
- Enrichment media

code código	ml ml	description descripción	case qty. cant. caja	case wt. peso caja	case vol. vol. caja	package presentación
304211	2 ml	Thioglycolat broth Caldo Tioglicolato	6 x 60	2.84	0.025	rack-case caja-rack

Cases per pallet · Cajas por palet: 32

SHELF LIFE / CADUCIDAD: 12 months/meses

LIM BROTH CALDO LIM

IVD

Selective enrichment medium for Group B Streptococcus, including *S. agalactiae*. The kit consists of a tube with conical bottom, skirted and screw cap, containing 2 ml LIM broth, and a standard flocked swab. Packed in a plastic flow pack with the basic information and instructions printed.

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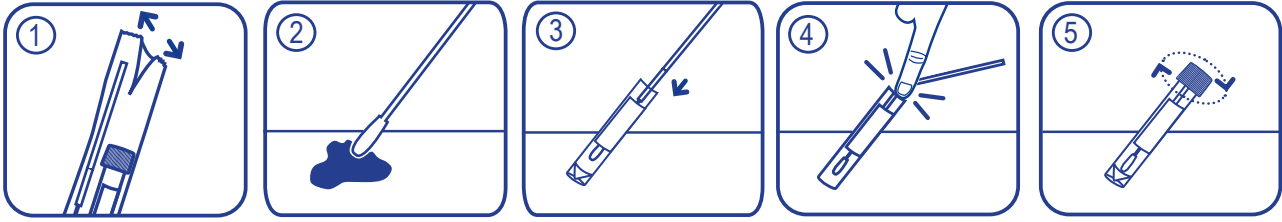
Packing Presentación

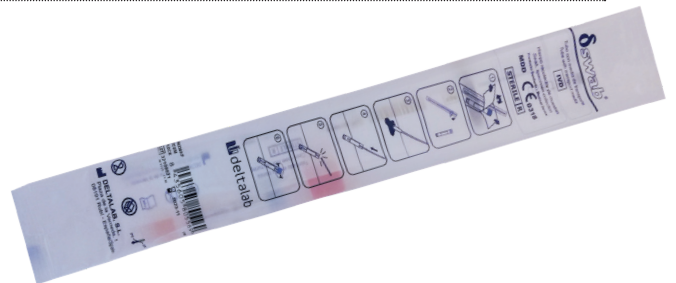
- ① The AMIES, CARY BLAIR and VIRUS are supplied in individual plastic flow-pack. Inside there is a swab and a tube with medium, both are sterile by radiation.

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- ② VICUM® transport media is supplied in a flow-pack made of plastic. Inside there is a swab inside a plastic peel-pack, sterile by radiation, and a tube with medium, sterile by filtration.

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- ③ The enrichment media SELENITE and TYOGLICOLATE are supplied in a box with internal rack to place 60 tubes.

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