

en

TONGUE DEPRESSOR















Only for single use. The reuse of this product may cause infections. Sterile by Ethylene oxide. Available in two materials:
 - Non-toxic and biocompatible wood, with shatterproof consistency and rounded edges to prevent injuries in the oral area.
 - Non-toxic plastic (polystyrene), biocompatible and with rounded edges to prevent injuries in the oral area.

With an appropriate mechanical consistency to guarantee the impossibility of breakage during its use. Used as tongue depressor to get correct views of the oral area and to prevent contamination during throat or nasopharyngeal sampling.
 Unit presentation: unitary packing in flow-pack.

The depressor is intended to be used in clinical and hospital environments. The use of the depressor presents no difficulty for the intended users. Not suitable for any other application than its intended use. Do not use if there are any signs of packaging breakage or damage. Do not use the depressor in case of signs of lack of integrity of the unitary packaging as it could have lost its sterility. Do not use if dirt is observed on the depressor. Discard the product if any broken or chipped depressor is found. Any strong pressure during use, although unlikely, could break the depressor. Do not use if the printed product information is not displayed correctly. Do not use after the expiry date.


In case of any damage, discard immediately. Depressors do not need special storage conditions. Due to the intended use of the product, the end users belong to the health sector and they have biological waste containers, where the used products should be discarded and managed as biological waste. The depressors, once used, must be decontaminated or treated as contaminated medical waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to the regulations. Unused depressors can be considered as non-hazardous products and can be discarded according to this criterion.

REF.	DESCRIPTION
441142.2	PLASTIC TONGUE DEPRESSOR I/W
441242.2	WOOD TONGUE DEPRESSOR FLOW

 Quantity	 CE marking 0318	 Batch code	 Manufacturer
 Catalogue number	 Do not use if Package is damaged	 Do not re-use	 Sterile by Ethylene Oxide
 Medical Device	 Consult instructions for use on the website www.deltalab.es/eifus	 Manufacturing date	 Use-by-date
 Unique device identifier	 Unique sterile barrier system		

In case of a serious incident* related to the product, notify to Deltalab, S.L. as well as the competent authority of the State in which the user is established.

*A "serious incident" is understood as one that entails the death, or serious deterioration of the health of the patient or user or a serious threat to public health.

DELTA LAB, S.L.
 Plaza de La Verneda 1, Pol. Ind. La Llana,
 08191 Rubí, Barcelona. España/Spain.
info@deltalab.es - www.deltalab.es