

en

## AYRE SPATULA

Product intended to facilitate the cervical sampling by healthcare personnel by specialized healthcare personnel and in the appropriate hygienic conditions.  
Single-use product. Ayre spatulas are medical device (MD).

### Intended use:

Product intended for scraping and extraction of cytological material from the surface of the cervix or vaginal area for pathological examination. Intended for use in clinical-hospital settings. They have a sufficient mechanical consistency to ensure the impossibility of breaking during use.










### Presentation:

Presentation in bulk. Available in two materials:

- High-quality wood, with a shatterproof consistency and rounded edges to avoid aggressiveness on mucous membranes and tissues.
- Plastic (Polypropylene) with rounded edges to avoid aggressiveness on mucous membranes and tissues.


### General precautions:

1. The use of the spatula does not present any difficulties for the intended users.
2. Not suitable for any application other than its intended use.
3. Reuse of this product may cause infections.
4. Do not use if dirt is visible on the spatula. Discard the product if any are found to be broken or chipped.
5. Strong pressure should not be exerted during use as the spatula may break (unlikely as the force required to break it is considerable). In case of breakage, discard immediately.
6. The spatulas do not require special storage conditions.
7. Due to the intended use of the product, the recipients belong to the healthcare sector, and have biological waste containers where the used product must be disposed of, and managed as biological waste. Spatulas, once used, must be decontaminated or treated as contaminated hospital waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to regulations. Unused spatulas can be considered non-hazardous goods and can be disposed of according to this criteria.

 Do not re-use	 CE marking	 Manufacturer
 Medical Device	 Consult instructions for use on the website <a href="http://www.deltalab.es/eifus">www.deltalab.es/eifus</a>	 Use-by-date
 Catalogue number	 Unique device identifier	 Batch code

In case of a serious incident\* related to the product, notify to Deltalab, S.L. as well as the competent authority of the State in which the user is established.

\*A "serious incident" is understood as one that entails the death, or serious deterioration of the health of the patient or user or a serious threat to public health.



DELTALAB, S.L.  
Plaza de La Verneda 1, Pol. Ind. La Llana,  
08191 Rubí, Barcelona. España/Spain.  
[info@deltalabgroup.com](mailto:info@deltalabgroup.com) - [www.deltalab.es](http://www.deltalab.es)