

STERILE CONTAINERS FOR SAMPLE COLLECTION

For collection, storage, and transportation of human clinical samples for subsequent examination.

Read product's instructions for use carefully before use.

Single use. Sterile product. The sterilization method is indicated on the primary packaging.

Product not made with natural rubber latex or dry natural rubber.

Medical device for In Vitro Diagnostic.

Intended use:

Sterile sample collection container (flasks, tubes or kits) designed for the collection, storage and transport of human clinical specimens (as urine, sputum, faeces, etc.). The product is intended to be used for the patient to collect the urine according to the instructions for previous preparation and sample collection indicated by the health personnel or laboratory.

General precaution:


- Not suitable for any application other than its intended use.
- Do not squeeze or press the container.
- Do not use if the product or any of its components are damaged or soiled as this may compromise the sterility of the product.
- Do not re-use. The re-use of this product may affect the subsequent analysis of the sample taken.
- Keep away from sunlight.
- Do not use if the expiry date has been exceeded.
- Do not use if the printed information is not displayed correctly.
- Do not use if packaging is damaged.

Special precautions:

Healthcare professionals must validate the use of the container for their specific assay-instrument/reagent system combinations and specimen storage conditions.

Instructions for the patient:

1. Wash your hands and dry with paper towel.
2. Remove protective wrapping (if applicable).
3. Verify that the container is tightly closed and that the safety closure, if present, has not been opened, as this could compromise the sterility of the product. Remove the lid from the container (if applicable) and place it upside down on a flat surface avoiding the inner part of the lid to contact anything, preventing its contamination.
4. Collects sample as per the facility's or facultative's instruction, considering previous preparation as indicated.
5. Deposit the specimen directly into the container and put on the lid again on the container (if applicable) to avoid contamination.
6. Return the container or tube to the healthcare professional after sample collection.

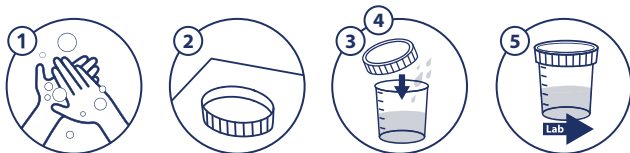

Instructions for transport of the sample:

1. All collection devices, both containing sample or used, should be classified as biohazardous for handling and disposal purposes. It is responsibility of each laboratory to handle, treat and dispose of waste according to current legislation. Non-used containers may be considered non-hazardous and may be disposed of according to these criteria.
2. Properly label containers with patient's name, i.d., collection date and time and any additional information required by your facility's policy.
3. Properly label and package any container used to transport specimen to alternate location in accordance with applicable local and state requirements.

Instructions for sample processing:

1. Follow standard precautions when testing the sample: wear gloves, lab coat, eye protection or other personal protective equipment to protect against potential sample splashes, leaks, or possible exposure to pathogens.
2. Treat the sample according to the protocols established by your center/laboratory.
3. Discard the containers for sample collection according to your center or laboratory's protocols/legislation for discard biohazardous residues.



References:

1. Nickander, K et al. Urine culture transport tubes: effect of sample volume on bacterial toxicity of the preservative. J Clin Microbiol. 1982, 15(4):593.
2. Clinical Laboratory Standards Institute (CLSI), Urinalysis - Approved Guideline – Third Edition, GP16-A3, Wayne, PA, 2009.
3. Cabedo C, et al. ¿Es importante la técnica de recogida de la orina para evitar la contaminación de las muestras?. Aten Primaria 2004;33(3):140-4. 2003.

Symbol glossary:

REF	Catalogue number	LOT	Batch number	www.deltalab.es/eifus	Consult instructions for use on the website	QTY	Quantity
IVD	In vitro Diagnostic	Do not re-use	Do not re-use	Do not use if package is damaged	Do not use if package is damaged	Precaution	Precaution
Keep away from sunlight	Keep away from sunlight	Manufacturer	Manufacturer	Use-by-date	Use-by-date	CE marking	CE marking
UDI	Unique device identifier	STERILE A	Sterile by aseptic processing techniques	STERILE R	Sterile by irradiation	STERILE EO	Sterile by Ethylene Oxide