

## AYRE SPATULA

Only for single use. The reuse of this product may cause infections. Sterile by Ethylene oxide. Available in two materials:

- Wood with shatterproof consistency and rounded edges to prevent injuries in mucus and tissues.
- Non-toxic plastic (polypropylene), with rounded edges to prevent injuries in mucus and tissues.

It has an appropriate mechanical consistency to guarantee the impossibility of breakage during its use. The product is intended to facilitate cervical sampling by specialised health professionals, and it is intended to be used in appropriate hygienic conditions. Presentation: unitary packaging in flow-pack or peel-pack.

The spatula is intended to be used in clinical and hospital environments. The use of the spatula presents no difficulty for the intended users. Not suitable for any other applications than its intended use. Do not use if there are any signs of packaging breakage or damage. Discard if the unitary packaging shows signs of lack of integrity, as it could have lost its sterility. Do not use if dirt is observed in the spatula. Discard the product if you find any broken or chipped spatula. Do not use if the printed product information is not displayed correctly. Do not use after the expiry date. Should not be exerted strong pressure during its use, otherwise the spatula might be broken (which is rather unlikely because the strength required to break it is considerable). If it is broken, discard immediately. The spatula does not need special storage conditions. Due to the intended use of the product, the end users are health professionals who have biological waste containers, where the used product should be discarded and handled as biological waste. The spatulas, once used, must be decontaminated or treated as contaminated medical waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to the regulations. Unused spatulas can be considered non-hazardous products and can be discarded according to this criterion.

REF.	DESCRIPTION
444150	WOOD AYRE SPATULA PEEL-PACK
444142	PLASTIC AYRE SPATULA PEEL-PACK
444242	PLASTIC AYRE SPATULA FLOW

## CYTOLOGY BRUSH















Only for single use. The reuse of this product may cause infections. Sterile by Ethylene oxide.

The intended use of this product is the collection of endocervical samples for laboratory studies (cytological, anatomopathological, microbiological, etc.). The shape and texture of the head provide a precise sample collection without damaging the mucous. The brush allows the user to take a sufficient sample quantity in case of more than one test. The intended use of this product is the collection of cervical samples by specialized sanitary professionals and under appropriate hygienic conditions. Do not use in case of pregnancy. Unit presentation: unitary packaging in peel-pack.

The Cytology Brush is intended to be used in clinical and hospital environments. The use of the cytology brush presents no difficulty for the intended end users. Not suitable for any other application than its intended use. Do not use it if there are any signs of packaging breakage or damage. If the unitary packaging shows signs of lack of integrity, do not use as it could have lost its sterility. Do not use if dirt is observed on the brush. Discard the product if any broken brush or without head are found. Do not use if the printed product information is not displayed correctly. Do not use after the expiry date.

Should not be exerted strong pressure during its use, otherwise the Cytology Brush might be broken (which is unlikely, because the strength required to break it is considerable). If it is broken, discard immediately. Cytology Brush does not need special storage conditions. Due to the intended use of the product, the end users are health professionals, who have biological waste containers, where the used product should be discarded and handled as biological waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to the regulations. Unused brushes can be considered nonhazardous products and can be discarded according to this criterion.

REF.	DESCRIPTION
440151	CYTOLOGY BRUSH PEEL PACK

 Quantity	 CE marking	 Batch code	 Manufacturer
 Catalogue number	 Do not use if Package is damaged	 Do not re-use	 Sterile by Ethylene Oxide
 Medical Device	 Consult instructions for use on the website <a href="http://www.deltalab.es/eifus">www.deltalab.es/eifus</a>	 Manufacturing date	 Use-by-date
 Unique device identifier	 Unique sterile barrier system		

In case of a serious incident\* related to the product, notify to Deltalab, S.L. as well as the competent authority of the State in which the user is established.

\*A "serious incident" is understood as one that entails the death, or serious deterioration of the health of the patient or user or a serious threat to public health.