

en

## CYTOLOGY BRUSH

**Product used to facilitate the cervical sampling by specialized healthcare personnel and in appropriate hygienic conditions, intended for laboratory studies (cytological, pathological, microbiological, etc.).  
Single-use product. Cytology brush are medical devices (MD).**

### Intended use:

Product intended for obtaining a cervical biopsy of a suspicious area or with visible exocervical lesions detected during a vaginal examination, in order to obtain tissue from a woman with intraepithelial disease.

Intended to be used in clinical and hospital environments.










The shape and texture of the head allow a sufficient amount of sample to be taken in case more than one test has to be carried out.

### Presentation:

Presentation in bulk.


### General precautions:

1. It should not be used in case of pregnancy.
2. Reuse of this product may cause infections.
3. The use of the cytology brush does not present any difficulty for its intended uses.
4. Not suitable for any application other than its intended use.
5. Should not be used if dirt is observed on the brush. Discard the product if any are found to be broken or headless.
6. Strong pressure should not be exerted during use, as the cytology brush may break (unlikely as the force required to break it is considerable). In case of breakage, discard immediately.
7. Cytology brush do not require special storage conditions.
8. Due to the intended use of the product, the recipients belong to the healthcare sector, and have biological waste containers, where the used product must be disposed of, and managed as biological waste. Cytology brushes, once used, must be decontaminated or treated as contaminated hospital waste. It is the responsibility of each laboratory to handle, treat and dispose of waste according to regulations. Unused brushes can be considered non-hazardous products and can be disposed of according to this criteria.

 Do not re-use	 CE marking	 Manufacturer
 Medical Device	 Consult instructions for use on the website <a href="http://www.deltalab.es/eifus">www.deltalab.es/eifus</a>	 Use-by-date
 Catalogue number	 Unique device identifier	 Batch code

In case of a serious incident\* related to the product, notify to Deltalab, S.L. as well as the competent authority of the State in which the user is established.

\*A "serious incident" is understood as one that entails the death, or serious deterioration of the health of the patient or user or a serious threat to public health.

DELTALAB, S.L.  
Plaza de La Verneda 1, Pol. Ind. La Llana,  
08191 Rubí, Barcelona. España/Spain.  
[info@deltalabgroup.com](mailto:info@deltalabgroup.com) - [www.deltalab.es](http://www.deltalab.es)